

lifetime limited warranty

Procedure For Obtaining Warranty Services

1. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.
2. The complete (entire) Walker must be presented (returned) for inspection and evaluation. If the product is found to have defective materials or workmanship, the remedy will be limited to replacement or repair of the part needed to correct the problem. There will be no charge for the replaced or repaired materials.
3. Walkers which have been serviced under this warranty protection, will be returned to the owner at no additional cost for freight or delivery.
4. Warranty does not include any labor charges which may be incurred in the replacement or repair of parts or their installation. Freight or delivery charges to the location where such warranty services will be performed will be at the owners expense.
5. All Winnie Walker products are identified by a Lot Number at the time of manufacture. Removal or defacing the Lot Number may void the warranty.

For Service, in or out of warranty, complete the following information now for ready reference in the event correspondence becomes necessary. It is also required that you attach a copy of the Bill of Sale, should you be requested to show proof of purchase.

Dealers Name:	_____
Address:	_____
Phone No.:	_____
Walker Lot No.:	_____

lifetime limited warranty

Warranty Period

1. Your Drive brand product has lifetime protection against defective materials and workmanship covering the frame of the walker.

One (1) Year protection against defective materials or workmanship on all other components, assemblies and parts.

Limitations and Exclusions

Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond the reasonable control of Drive Products, are not covered by this warranty.

Parts or materials which are subject to normal wear resulting from the use of these products and which must be replaced/ repaired due to such normal wear, are the owners responsibility and are not covered by this Warranty.

This Warranty is exclusive and in lieu of all other express warranties, implied warranty, including but not limited to the implied warranty of merchantability and fitness for a particular purpose, and shall not extend beyond the duration of the express warranty provided herein. Drive Medical Design & Manufacturing shall not be liable for any consequential or incidental damages whatsoever.

This Warranty gives you specific legal rights which may vary from state to state. Some states do not allow the exclusions or limitations of incidental or consequential damages, or how long an implied warranty lasts, so the exclusions and limitation may not apply to you.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

mimi lite
winnie walker

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

mimi lite

At only 11 lbs. the Mimi Lite is the lightest four wheeler available. The push down lock system is simple to operate, making this unit very user friendly. The easy folding and lightweight features combine to make this a great travel Rollator.

parts

- (2) handlebar
- (2) height adjustment knobs with bolts
- (1) backrest
- (1) tote bag
- (1) folding strap

assembly instructions

Please read these instructions before assembly or use of this Rollator. This Rollator must always be assembled and fitted for the user by a qualified Drive Medical Authorized dealer. Before using your new Rollator please ensure that your weight is within the weight capacity of this unit. This Rollator will require very little assembly and the assembly can be completed without tools.

1. Open your Rollator. Push the folding hinge until straight to secure the unit in the proper position.
2. Place the open loop of the folding strap over the tube of the backrest assembly.
3. Slide the backrest assembly into the open receptacles on the front (locks are on the rear) of your new Rollator. Make sure that both sides of the backrest are firmly in place all the way at the bottom of each receptacle. Secure the backrest into place using the nylon screws included on the unit.
4. Place a handle bar with the pre-assembled Locking Loop Locks in each handle bar receptacle and adjust to the desired height.
5. Use the height adjustment knobs and bolts to secure the handle bars in this position being cautious to ensure that the bolt heads are properly mounted within the hex opening on the inside of each handle bar receptacle.
6. Test and adjust the lock mechanism tension to ensure proper and secure braking before use.
7. The black tote bag simply attaches to the frame of the unit beneath the seat. When properly positioned, the tote bag is open, directly under the seat when the frame is open.

push down lock

OPERATION:

Always engage and disengage wheel locks on both sides simultaneously.

TO SLOW DOWN OR COME TO A TEMPORARY STOP –
Push down slowly by applying weight on the handles until unit slows down or comes to a temporary stop.

TO TEMPORARILY LOCK WHEELS IN POSITION –
Once seated, rear wheels will automatically lock until weight is removed.

TO RELEASE WHEEL LOCKS –
Remove weight until wheels start moving freely.

folding

This unit is easily folded from the user's side of the unit by grasping the folding strap (black cloth strap) located under the hinged seat and lifting the unit by this strap. The unit will fold as lifting for easy storage and transport. This unit may be folded with the tote bag in place.

cautions

Please read all cautions and warning labels before using this Rollator.

This unit is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight distribution between the handlebars with the unit in front of the user.

If Rollator has a seat, always engage the locking mechanism on the push down locks before using the seat. At no time can this unit be used to transport individuals while seated for risk of bodily injury.



Item #510 Shown
Push Down Lock