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Instructions for Use – Please Read All Instructions Before Use. Setup

- Remove frame, seat, lid, pail, cover, and splash guard from carton.
- Attach seat and then lid to rear support of frame. Place slash guard or pail and cover in frame.
NOTE: When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.
NOTE: Push down firmly until the seat snaps into place.
NOTE: DO NOT use excessive force when installing the seat and lid.

Adjusting Leg Height

- Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible “click” will be heard.
- Repeat for other 3 legs.
CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Inserting Back Bar

- Remove back bar from carton.
- Insert back bar into channel guides on back of commode until back bar push buttons protrude fully through channel guide holes (you will hear an audible click).

Use Commode over Toilet

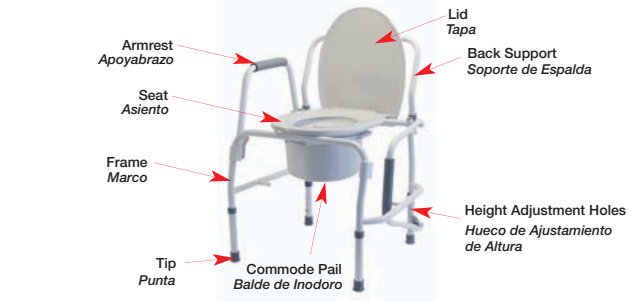
- Set commode to desired height (described in previous steps)
- Remove commode pail and lid.
- Position splash guard in place onto pail supports
- Remove back bar (described in previous steps)
- Lift toilet bowl seat
- Place commode over toilet with slash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

Use as Bedside Commode

- Set commode to desired height (described in previous steps)
- Lift commode seat and lid and place commode pail onto pail supports. Ensure that pail is properly positioned below commode opening.
- Place commode seat down. Commode is now ready for use.
- When use is finished using commode, place commode pail lid over commode pail to contain odors.
- Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

Arm Adjustment

- Drop Armrest:** to drop the armrest, push on the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

Features

- Arms drop independently below seat level for safe lateral transfers
- Easy-to-release drop arm mechanism may be operated by patients lacking manual dexterity
- Arms automatically lock in raised position
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame
- Welded steel construction increases strength and stability
- Cushioned armrests enhance user comfort
- Easy to attach plastic seat [6433A, 6433A-2]
- Easy to attach padded seat [6437A]
- Comes complete with commode pail, cover and splash guard
- Maximum weight capacity of 300 lb
- Designed to meet the requirements of HCPCS code: E0165

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© 2009, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360.
Made in China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

GF0500036RevC09

Silver Collection Steel Drop Arm Commode

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Baja Apoyabrazo

Items 6433A, 6433A-2 & 6437A

User Instructions and Warranty Information

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Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex products listed on the cover of this manual for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care

Parts covered under the lifetime warranty include: (frame, welding, screws, frame locking components). Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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