



Helping all people  
live healthy lives

# BD Eclipse™ Needle

# How to Use the BD Eclipse™ Needle for Luer Lock Syringes

## 1. Attachment

- 1a. Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.
- 1b. Pull back on the safety cover toward the syringe and away from the needle.
- 1c. Grasp the syringe with one hand and with the other hand pull the clear needle shield straight off.



## 2. Aspiration

Draw up medication as per your usual technique.



**NOTE:** If changing needles, activate safety cover prior to removing the needle from the syringe.

## 3. Injection

Administer medication in accordance with established protocol. For user convenience, the needle “bevel-up” position is oriented to the safety cover.



## 4. Activation

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward toward the needle until you hear and/or feel it lock. Visually confirm that the needle tip is covered.




**Discard after single use in an approved sharps container.**

## Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

# The BD Eclipse™ Needle offers clinicians a wide range of features and benefits for luer lock applications.

## Prior To Activation

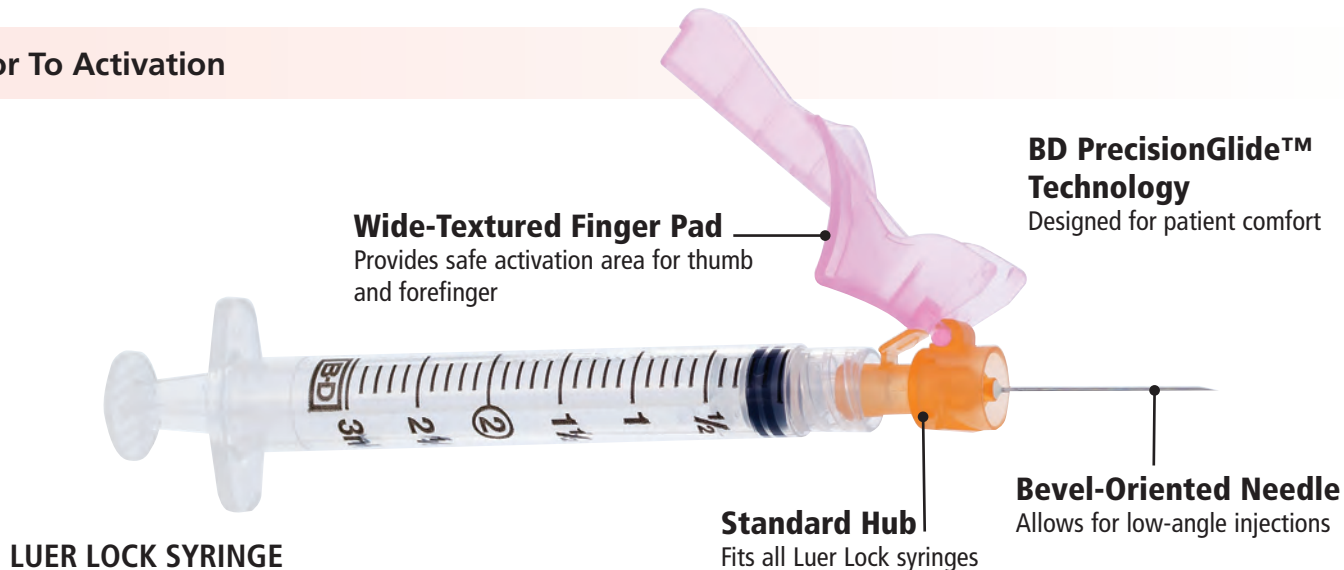

## After Activation



# Designed With Important Safety Features And Backed By BD's Reputation For Quality

## BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

## Safe, One-Handed Activation

- Single finger stroke activation:  
no hard surface required
- Textured finger pad keeps fingers  
behind the needle
- Requires minimal change in  
injection technique

## Designed To Protect Healthcare Workers From Accidental Needlestick Injuries

- Safety cover is present between clinician  
and needle
- Audible click indicates safety activation
- Needle remains locked inside activated cover
- The BD Eclipse™ Family offers one safety  
technology for both your injection and  
blood collection needs



**BD Eclipse™ Needle for Luer Lock Syringes - 100/Box – 1200/Case**

Catalog #	Needle Size	Catalog #	Needle Size
305761	25G x 1"	305765	21G x 1½"
305767	25G x 1½"	305766	18G x 1½"
305762	23G x 1"	305759	25G x ⅝"
305768	22G x 1"	305758	27G x ½"
305763	22G x 1½"	305757	30G x ½"
305764	21G x 1"		

**BD Eclipse™ Needle with Luer Lock Syringe  
1mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case**

305778	30G x ½"	305780	25G x ⅝"
305789	27G x ½"		

**3mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case**

305781	25G x ⅝"	305783	22G x 1½"
305787	25G x 1"	305779	21G x 1"
305782	23G x 1"	305784	21G x 1½"
305788	22G x 1"		

**5mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case**

305785	22G x 1 ½"		
--------	------------	--	--

**10mL BD Luer -Lok™ Syringe with Detachable Needle - 50/Box - 300 Case**

305786	22G x 1½"		
--------	-----------	--	--

**REMEMBER:** Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call  
**1.888.237.2762**



**BD Medical**  
1 Becton Drive  
Franklin Lakes, NJ 07417

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD.

BD 10/13-MSS0078 - Rev1

[www.bd.com](http://www.bd.com)