

## Purpose

- A drainable pouch is open at the bottom to allow emptying. It requires a closure on the bottom to keep it closed.
- A pouch should be emptied when 1/3 to 1/2 full of stool or gas.
- The Lock 'n Roll Microseal Closure is an integrated closure on a drainable pouch.

## Using the Lock 'n Roll Closure


### To Open and Empty the Pouch



1. Lift the security flap towards you.



2. Unroll pouch tail three times.

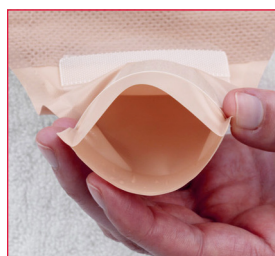

3. Pinch the pouch tail open and allow pouch contents to empty into a toilet or receptacle.



4. Clean pouch tail with a tissue or moist towelette.

### To Close the Pouch



1. Roll the pouch tail up three times towards yourself until the strip of interlocking fasteners is facing upwards.



2. Align the strip of fasteners and press to the outer edges to seal the interlocking fasteners.

**Q: How do I know my pouch is securely fastened?**

**A:** Aligning and pressing the center of the security flap to the outer edges will allow you to feel the interlocking fasteners seal and ensure your pouch is securely closed.

**Q: Will the closure work when it is wet?**

**A:** The material used on the closure is water-resistant and can be securely closed even when it is wet.

**Q: Will the closure show through my clothes?**

**A:** The closure is designed to be discreet with its slim profile that is not noticeable underneath most clothes.

**Q: Is the closure comfortable against my body?**

**A:** The slim profile of the ergonomically curved outlet is designed to conform to the body's natural curves providing the utmost comfort.

**Q: Will the closure open accidentally?**

**A:** Laboratory testing has shown that the interlocking fasteners are secure when closed properly.

**Q: My output is very liquid. Are there any suggestions to make emptying easier?**

**A:** With the closure's unique design, you may want to control the draining of the liquid contents by unrolling the pouch tail two times and holding the last flap up. You then can unroll this flap slowly.

**Q: Will people hear me open the pouch when I need to empty it in a public rest room?**

**A:** The material used on the closure is quiet and will not make a "ripping" noise when the security flap is opened.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **1.888.740.8999**. In Canada call **1.800.263.7400**.



**Hollister Incorporated**  
Libertyville, IL USA

Distributed by  
**Hollister Limited**  
Aurora, Canada

[www.hollister.com](http://www.hollister.com)