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LIMITED WARRANTY

American Diagnostic Corporation (ADC®) warrants its products against defects in materials and workmanship under normal use and service as follows:

1. Warranty service extends to the original retail purchaser only and commences with the date of delivery.
2. The optical system is warranted for one year.

What is Covered: Repair, or replacement of parts, and labor.

What is Not Covered: Transportation charges to and from ADC®, Damages caused by abuse, misuse, accident, or negligence. Incidental, special, or consequential damages. Some states do not allow the exclusion or limitation of incidental, special, or consequential damages, so this limitation may not apply to you.

To Obtain Warranty Service: Send item(s) postage paid to ADC®, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, a brief note explaining the problem, and \$2.00 to cover the cost of return shipping and handling.

Implied Warranty: Any implied warranty shall be limited in duration to the terms of this warranty and in no case beyond the original selling price (except where prohibited by law). **This warranty gives you specific legal rights and you may have other rights which vary from state to state.**

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Environment, Environment, Ambiente

Transportation / Storage:	-20°C to +49°C (-4°F to 120.2°F) RH 95 % 500 hpa – 1060 hpa, Altitude
Environmental Operating Range:	10°C to 49°C (50°F to 120.2°F) RH 95 % (Max) 500 hpa – 1060 hpa, Altitude
Max. Operating:	35°C (95°F)

Symbols, Symboles, Sybolos

Attention. Read Operating Manual for Cautions and Instructions for Use.
Attention. Lisez les avertissements et le mode d'emploi du manuel d'utilisation.
Atención. Lea las "Precauciones" e "Instrucciones de uso" del manual de funcionamiento.

Attention. This is NOT an AP or APG apparatus

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Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by Directive 2002/96/EC of the European Parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE). If this product is contaminated, this directive does not apply.

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To Register Your Product, visit us at **www.adctoday.com** and follow the links

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ADC
55 Commerce Drive
Hauppauge, NY 11788

ADC (UK) Ltd.
Unit 6, PO14 1TH
United Kingdom

Inspected in the U.S.A.
Made in Dubai
tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com
info@adctoday.com

Thank you for choosing an ADC® Diagnostic set. We're proud of the care and quality that goes into the manufacture of each and every diagnostic instrument that bears our name. Every component has been carefully designed to maximize performance. This booklet refers to both Otoscope and Ophthalmoscope sets (Models 5110N, 5110E, 5111N, 5112N).

Device Description and Intended Use

An otoscope is a handheld device that provides illumination of the ear canal for observation by using a battery powered light source and an optical magnifying system.

An ophthalmoscope is a handheld battery powered device containing illumination and viewing optics intended to examine the media (cornea, aqueous, lens, and vitreous), the retina, blood vessels, optic nerve and other structures of the eye.

Contraindications

The use of this device is contraindicated in patients who have already received prolonged or intense light exposure, especially if the patients are infants, aphakes, or persons with diseased eyes. Excessive exposure to light may result in patient injury. See warnings for additional information.

General Warnings

A warning statement in this manual identifies a condition or practice which, if not corrected or discontinued immediately, could lead to patient injury, illness, or death.

CAUTION: Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. This device should be used with filters that eliminate UV radiation (< 400 nm) and, whenever possible, filters that eliminate short-wavelength blue light (<420 nm).

The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

While no acute optical radiation hazards have been identified for direct or indirect ophthalmoscopes, it is recommended that the intensity of light directed into the patient's eye be limited to the minimum level which is necessary for diagnosis. Infants, aphakes and persons with diseased eyes will be at greater risk. The risk may also be increased if the person being examined has had any exposure with the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography.

CAUTION: Do NOT immerse the battery handle in liquid or attempt to sterilize it in any way. Consult the care and maintenance instructions for each part for further information about sterilizing this product. Never clean or sterilize any device with batteries in it.

CAUTION: Federal law restricts this device to sale by or on the order of a physician or licensed health care practitioner.

CAUTION: When replacing lamp, allow lamp to cool before handling.

WARNING: Use of any accessories or materials not indicated in the user's manual can degrade the minimum safety of the equipment.

WARNING: Only replace listed accessories (attachments), lamps, or batteries for the device.

Otoscope Parts and Assembly

The Pocket Otoscope consists of a "AA" battery handle (#5160N), fiberoptic Otoscope, with attached movable viewing lens, and disposable specula (2.75mm and 4.25mm).

Turning Instrument On

To illuminate, slide the on/off switch located on the battery handle pocket clip away from the instrument head. Slide the switch in the opposite direction to turn off.

Attaching the Specula

Push the flanged end of the desired speculum onto the chrome socket and twist counterclockwise to engage. Reverse procedure to remove and discard after each use.

Viewing Lens

To move during an examination, turn the lens bezel clockwise to lift up. To reassemble reverse procedure.

Pneumatic Otoscope

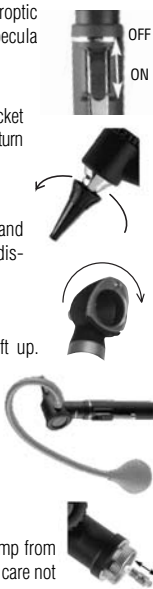
To perform, insert tapered end of insufflator fitting (#5121) sold separately, into hole on side of Otoscope head. Attach optional bulb assembly (#5122) sold separately.

Replacing Halogen Lamp

Remove the Otoscope head by turning counterclockwise. Grab end of lamp from inside base of Otoscope head and remove. Replace with new bulb, taking care not to touch glass.

Battery Replacement

This unit requires 2 "AA" batteries. For best performance, we suggest alkaline batteries. To replace, remove the base of the handle by turning counterclockwise and replace batteries, taking care to observe polarity. NOTE: This instrument should be turned OFF when not in use. If storing for extended lengths of time, remove batteries completely.

Ophthalmoscope Parts and Assembly

The Pocket Ophthalmoscope consists of a "AA" battery handle (#5160N) and Ophthalmoscope instrument head with halogen illumination.

Turning Instrument On

To illuminate, slide the on/off switch located on the battery handle pocket clip away from the instrument head. Slide the switch in the opposite direction to turn off.

Aperture Selection

This unit is equipped with 5 internal apertures - small, large, semi-circle, red-free, and fixation. To select, rotate the aperture selection wheel at the front of the instrument head.

Corrective Lens Selection

This unit is equipped with 19 corrective lenses from +20 to -20 diopters (negative lenses in red). To select, rotate the corrective lens selection wheel at either side of the instrument head. Selected lenses can be viewed in the panel.

Replacing Halogen Lamp

Remove the Ophthalmoscope head by turning counterclockwise. Grab end of lamp from inside base of Ophthalmoscope head and remove. Replace with new bulb, taking care not to touch glass, and align lamp metal guides on side of the lamp with grooves in lamp socket.

Battery Replacement

This unit requires 2 "AA" batteries. For best performance, we suggest alkaline batteries. To replace, remove the base of the handle by turning counterclockwise, and replace batteries, taking care to observe polarity. NOTE: This instrument should be turned OFF when not in use. If storing for extended lengths of time, remove batteries completely.

Care and Maintenance

The otoscope, ophthalmoscope, and battery handles may be cleaned externally with a moist cloth. In addition, the following disinfectants may be used for disinfection: Aldehyde (formaldehyde, glutaraldehyde, aldehyde derivatives), or surfactants. All instrument components, with the exception of the magnifying glass, and lens wheel, may be disinfected with alcohol. When using these substances, the manufacturer's instructions must be strictly complied with. Means for cleaning or disinfection may be a soft, possibly lint-free cloth or Q-tips.

WARNING: Never immerse the otoscope or ophthalmoscope in fluid. Ensure that no fluid penetrates into the interior of the enclosure.

Sterilization: According to teaching, Sterilization is only specified for surgery. Being that the otoscope, ophthalmoscope and battery handle are not being used in surgery, cleaning or disinfecting will be fully adequate.

Reusable ear specula: Although Sterilization is not required, the ear speculum may nevertheless be sterilized at 273.2° F (134° C) for a dwell time of 10 minutes in a steam sterilizer.

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