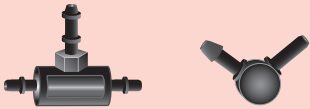


CUFF CONNECTIVITY

Congratulations on the purchase of your new ADC® replacement cuff for digital blood pressure home care units. You may have received 2 connectors. If so, use the diagram below to choose the correct connector to be used with your specific BP monitor. When connecting tubes to the appropriate connector, use a slight twisting motion and make sure tube and connector are firmly seated. Use the same slight twisting motion when inserting connector into monitor.

IMPORTANT: *Using the wrong connector for your specific monitor may lead to measurement error. Refer to your Blood Pressure unit's Instruction Booklet for cuff connection specific to your model.*

MONITOR MODEL	CONNECTOR	CONNECTORS	
6012, 6012N	897CN-1		<p>897CN-1</p> <p>897CN-2</p>
6013	897CN-2		
6014	897CN-2		
6017	897CN-2		
6021, 6021N	897CN-2		
6022, 6022N	897CN-2		
6023, 6023N	897CN-2		

Warning: This Product May Contain A Chemical Known To The State of California To Cause Cancer, Birth Defects, Or Other Reproductive Harm.