


SLIM LINE  
**EXAM TABLE**

WITH AN ADJUSTABLE BACKREST AND FULL BASE

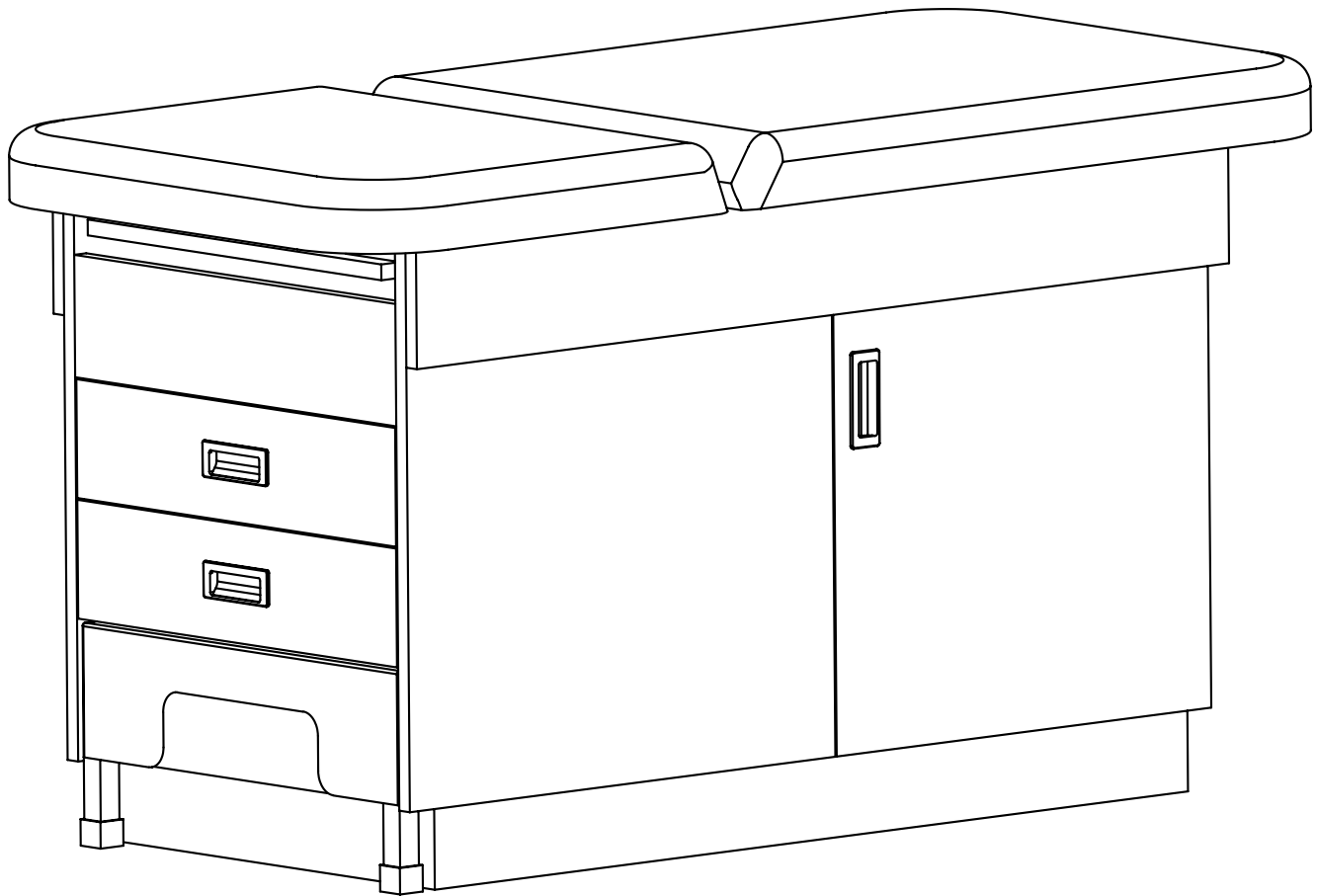

ADI996-FBASE-SERIES

**USER GUIDE**



# Thank you for your purchase.

---

**Viva Comfort maintains a strong practice of ongoing development of products and commercial research.**








---

We recognize that starting and maintaining a medical practice is expensive. We'll help make the process cost-effective and elegant. Additionally, our team of specialists will ensure everything you need works as efficiently as possible, so you can effectively treat your clients.




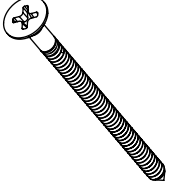
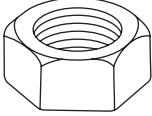
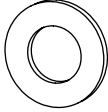
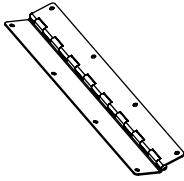
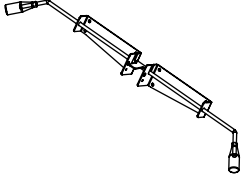
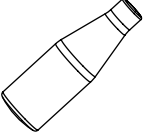
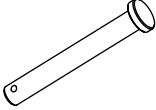

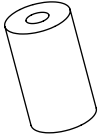
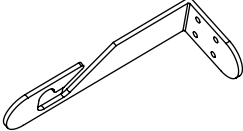
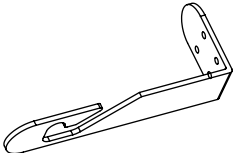
# Table of Contents

- 2 Parts
- 3 Hardware
- 4 Assembly Instructions
- 13** Installing Paper Roll
- 14** Warnings
- 15** Warranty


# Parts
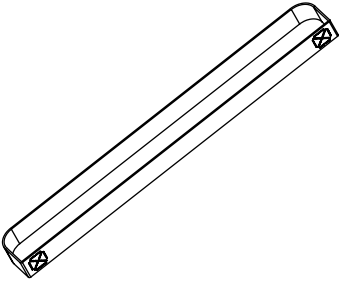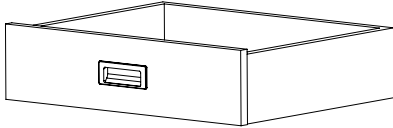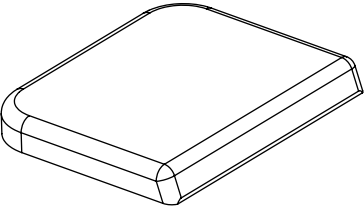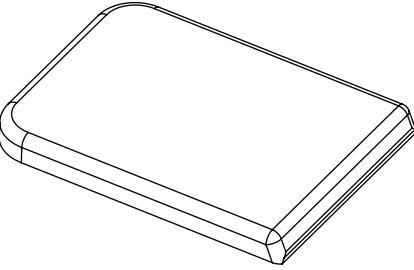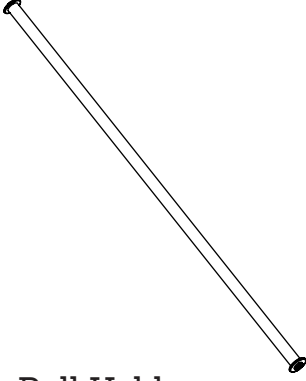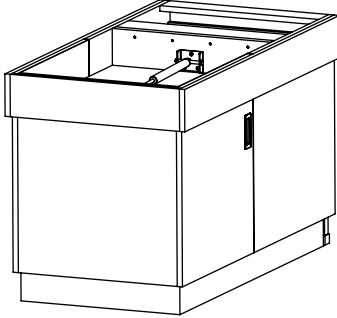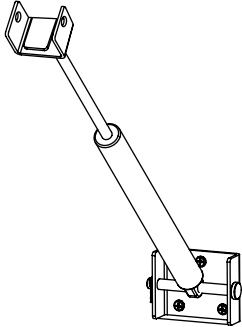
<p><b>A</b></p>  <p>Paper Cutter</p>	<p><b>B</b></p>  <p>Drawer (x2)</p>
<p><b>C</b></p>  <p>Small Mattress</p>	<p><b>D</b></p>  <p>Larger Mattress</p>
<p><b>E</b></p>  <p>Paper Roll Holder</p>	<p><b>F</b></p>  <p>Table Base</p>
<p><b>G</b></p>  <p>Pneumatic Lifting Rod</p>	

# Hardware
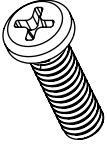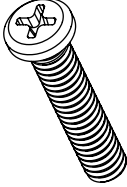
<p>1</p>  <p>M6* 20 Screw (x12)</p>	<p>2</p>  <p>M6*30 Screw (x10)</p>	<p>3</p>  <p>Tapping Screw (PA4*20) (x8)</p>	<p>4</p>  <p>Tapping Screw (PA6*20) (x2)</p>
<p>5</p>  <p>Nut (x2) (pre-installed)</p>	<p>6</p>  <p>Shim (M6) (x4)</p>	<p>7</p>  <p>Hinge</p>	<p>8</p>  <p>Lifting Auxiliary Rod (x1)</p>
<p>9</p>  <p>Handle (x2)</p>	<p>10</p>  <p>Latch</p>	<p>11</p>  <p>Snap Ring</p>	<p>12</p>  <p>Rubber Foot Pad (x2)</p>
<p>13</p>  <p>Left Support Element</p>	<p>14</p>  <p>Right Support Element</p>		

## Tools Needed (But Not Included):

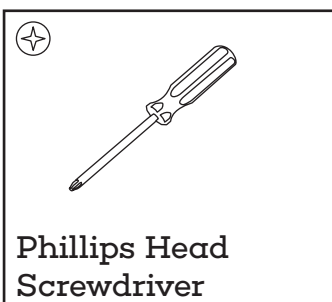


### NOTE:

We highly recommend using two people to complete the assembly of your Exam Table.

# Assembly Instructions


## Step 1:

- A. Lock the non-bent side of the Hinge (7) at the position shown below.
- B. Place four Shim's (6) over the predrilled holes on the Hinge (7) and thread four M6\*30 Screws (2) through the holes.
- C. Tighten the screws with a Philips head screwdriver.

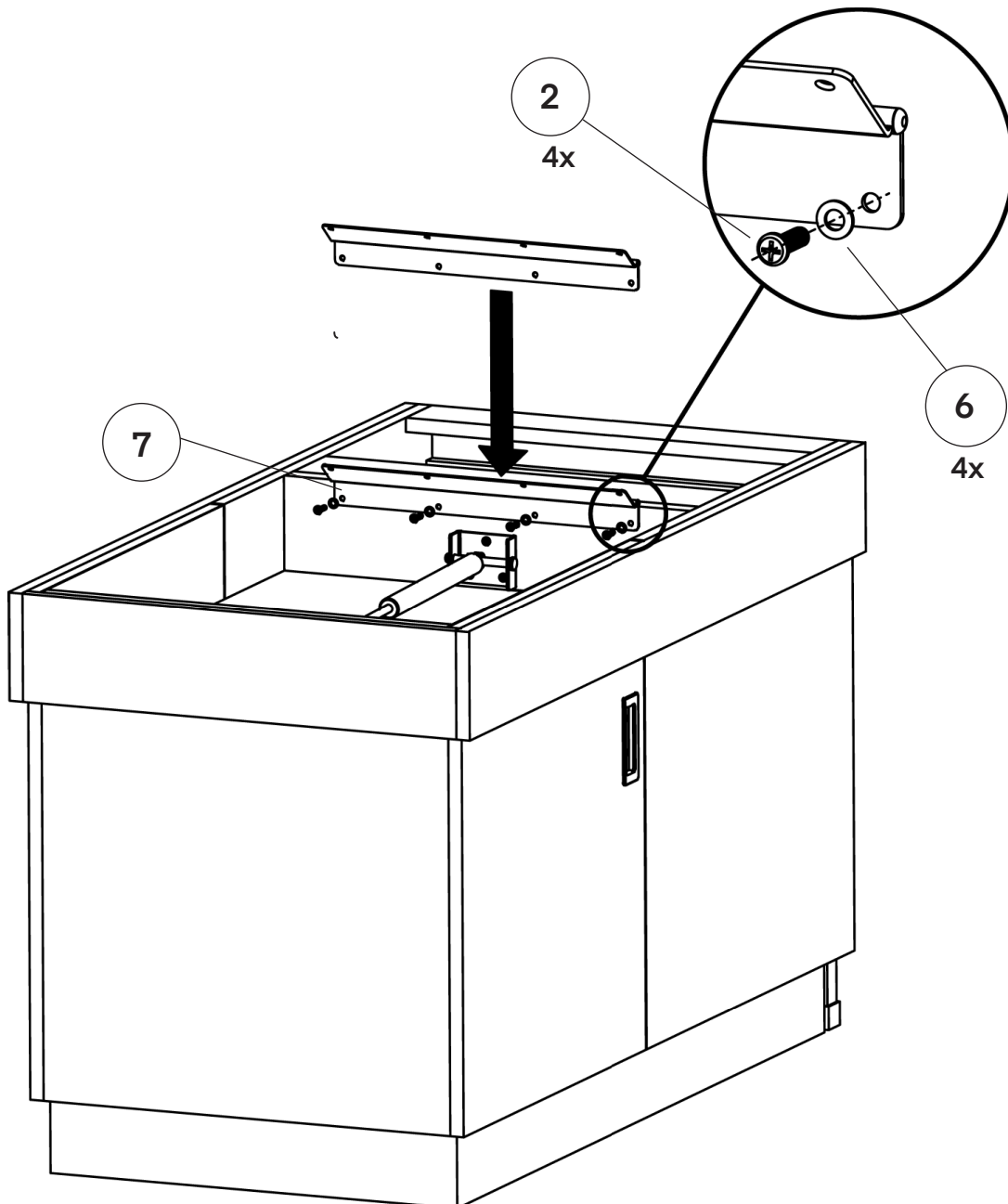


## Step 2:

- A.** Take your Larger Mattress (D) and put it on top of your Table Base (F), as shown.
- a.** Have someone hold it up in the angled position shown.
  - b.** You need to keep the Hinge (7) in the bent position shown so that you can affix it to the Larger Mattress (D).
- B.** Thread four M6\*20 Screws (1) through the holes on the Hinge (7), and into the Larger Mattress (D).
- C.** Secure the screws in place using a Philips head screwdriver.

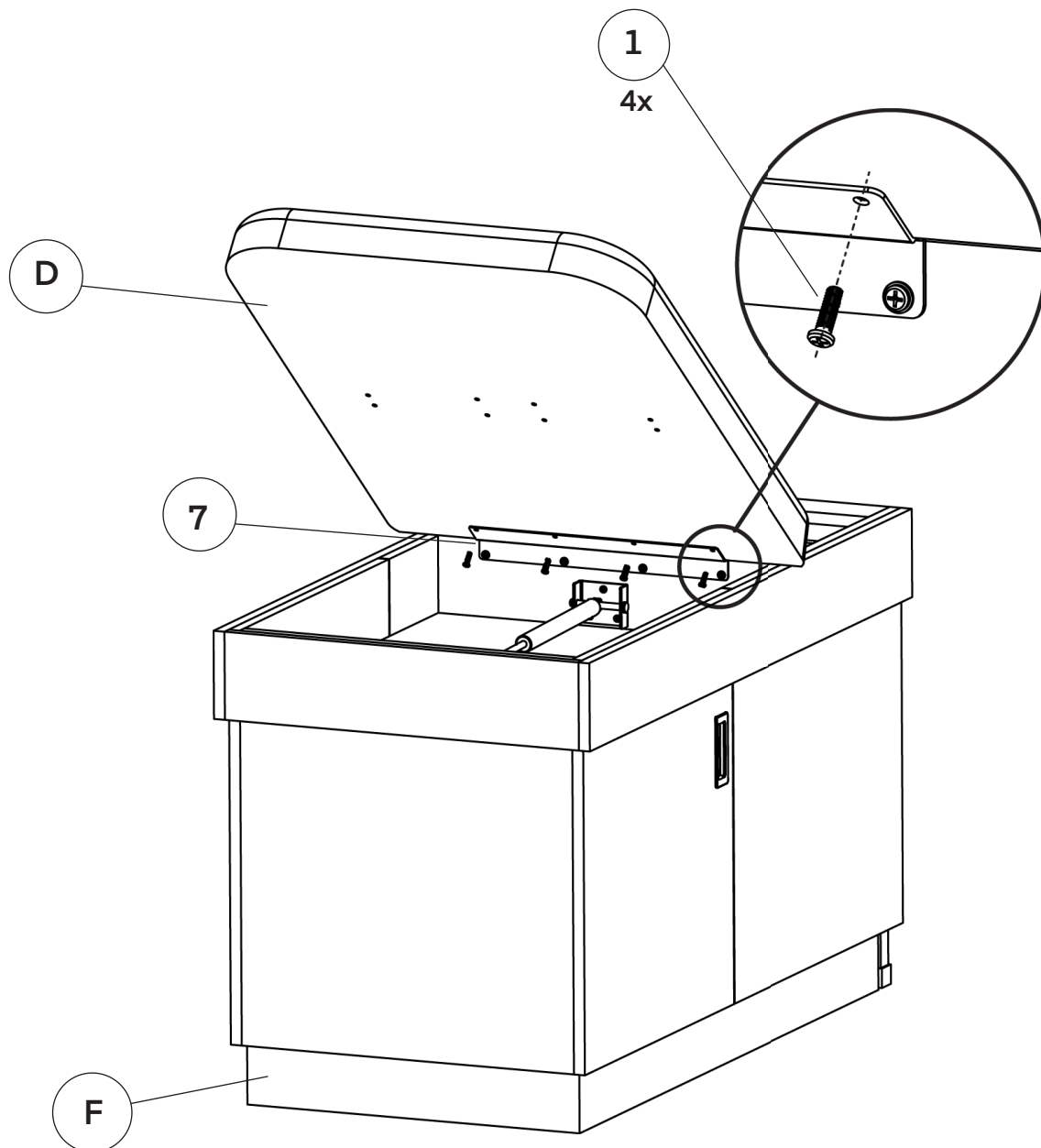


### Step 3:

- A. Locate the preinstalled Pneumatic Lifting Rod (G), the Lifting Auxiliary Rod (8), the Snap Ring (11), and the Latch (10).
- B. Remove the preinstalled Nuts (5), located on the Lifting Auxiliary Rod (8).
- C. Lift the Pneumatic Lifting Rod (G), place the Lifting Auxiliary Rod (8) between the bar on the Pneumatic Lifting Rod (G) as pictured.
- D. Put the Latch (10) through the holes on both the Pneumatic Lifting Rod (G) and Lifting Auxiliary Rod (8), as pictured.
- E. Secure the Latch (10) in place with the Snap Ring (11).
- F. Insert both Handles (9) onto the ends of the Lifting Auxiliary Rod (8), until secured in place.

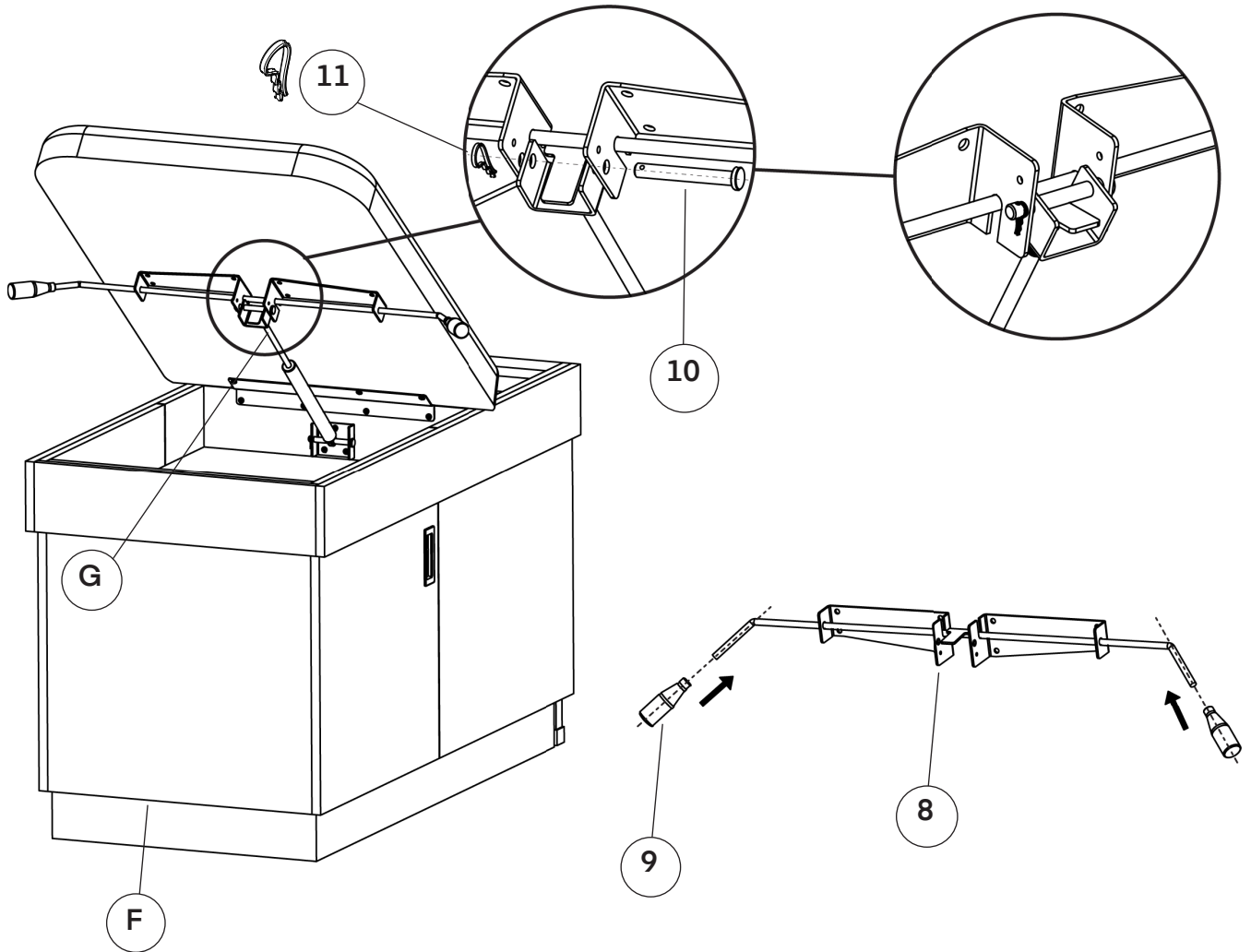


## Step 4:

- A. Align the Lifting Auxiliary Rod (8) with the holes on the Larger Mattress (D).
- B. Insert eight M6\*20 Screws (1) through the Lifting Auxiliary Rod (8) and into the Larger Mattress (D).
  - a. Secure them in place with a Philips head screwdriver.

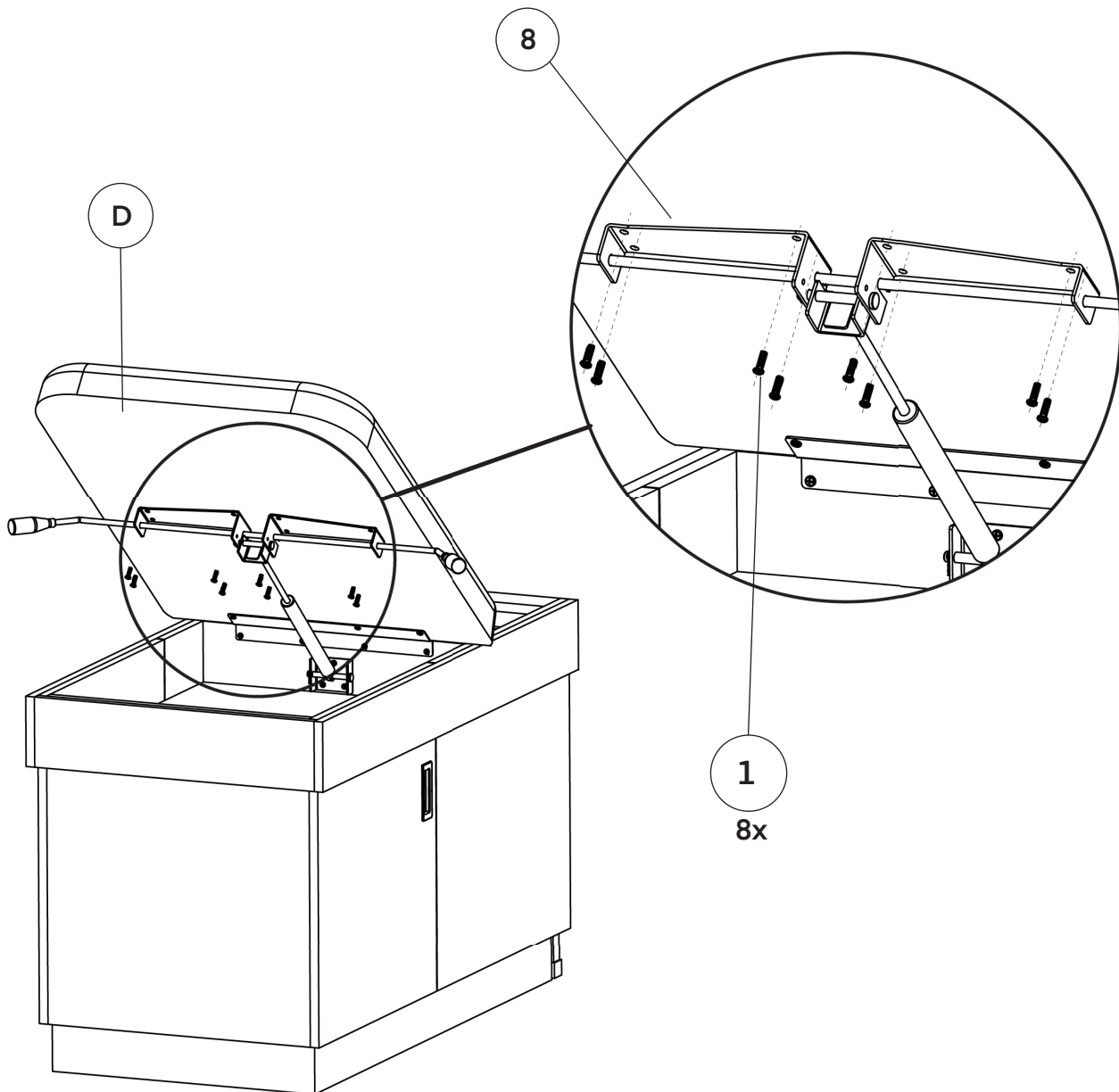


## Step 5:

- A. Located on the Pneumatic Lifting Rod (G) you must install two Nuts (5), as pictured.
- B. Adjust your pressure bar to the correct position and then tighten both Nuts (5).
  - a. Be mindful to leave space between both Nuts (5).

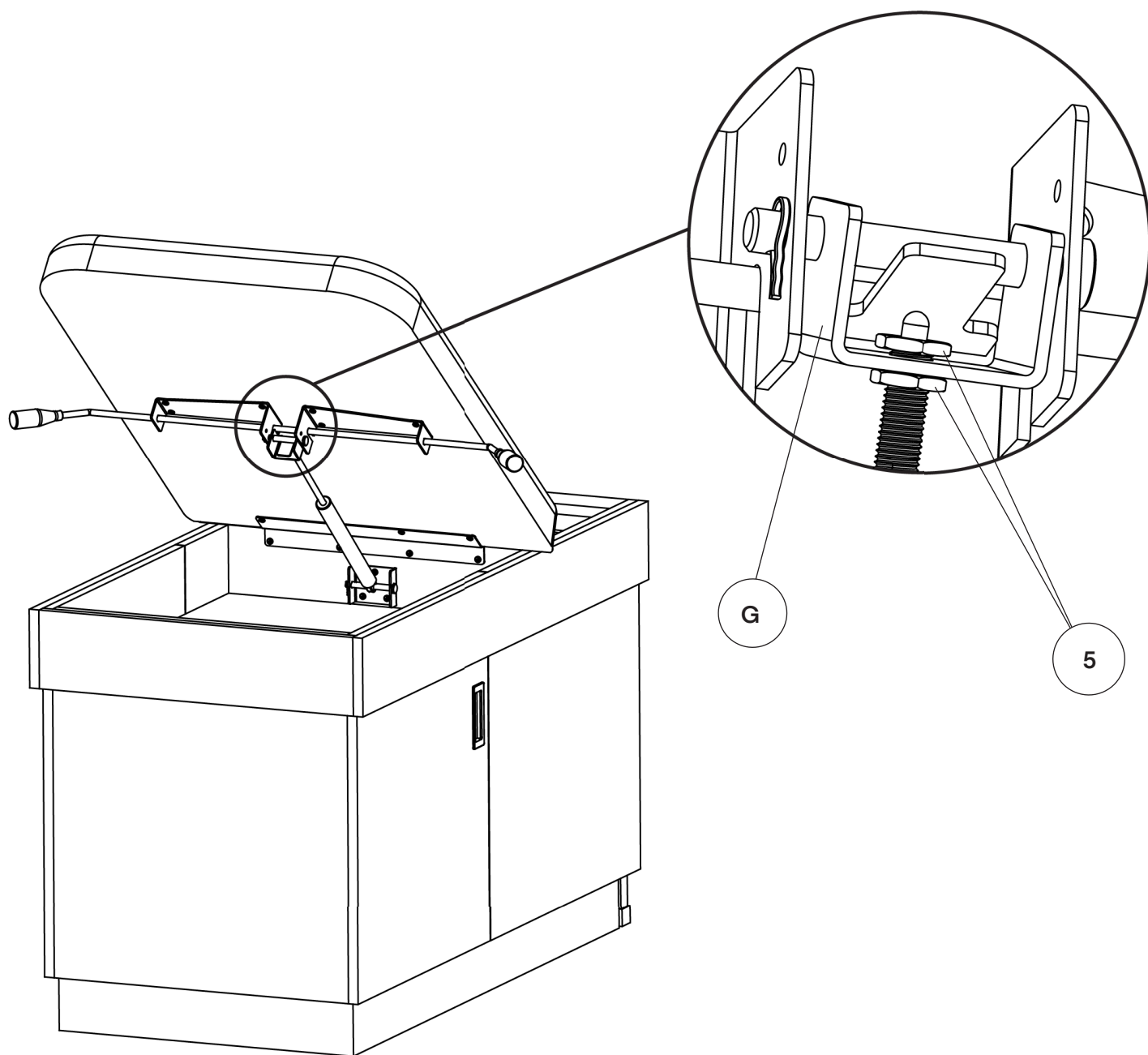


## Step 6:

- A.** Locate your Left Support Element (13), Right Support Element (14), Tapping Screw (PA4\*20) (3), and Tapping Screw (PA6\*20) (4).
- B.** Position your support elements on the Larger Mattress (D), as pictured and secure them in place with eight Tapping Screws (PA4\*20) (3), using a Philips head screwdriver.
- C.** Locate and place both Rubber Foot Pads (12) in the position shown below.
- D.** Secure them to the Larger Mattress (D) with two Tapping Screws (PA6\*20) (4), using a Philips head screwdriver.
- E.** Locate and insert the Paper Roll Holder (E) on the Left Support Element (13) and Right Support Element (14), as pictured.
- F.** Install the Paper Cutter (A) between the Rubber Foot Pads (12) and the Paper Roll Holder (E). Secure it in place as shown.

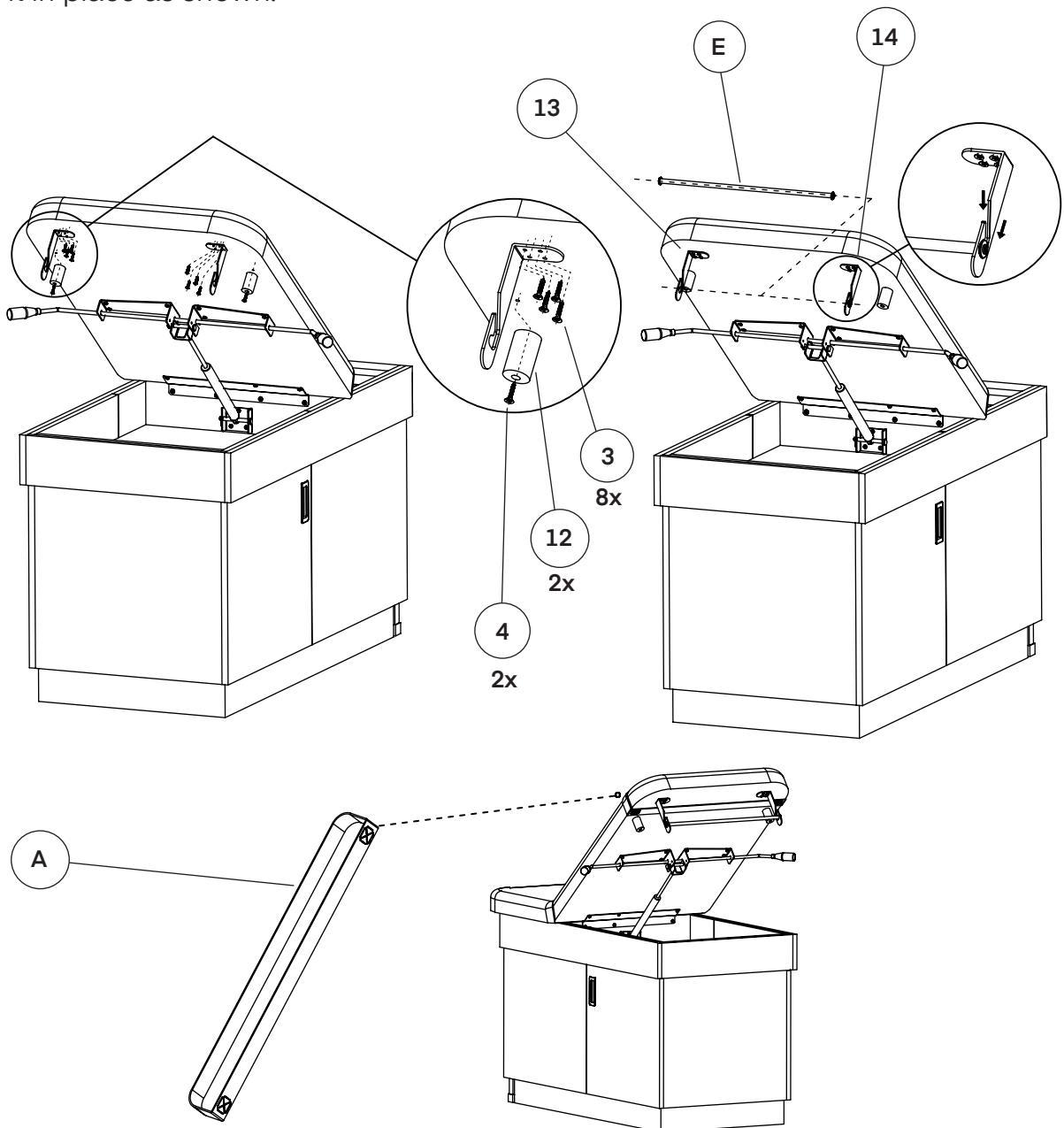


## Step 7:

- A. You will need to remove both Drawers (B).
- B. To do so, pull out the Drawer (B) and locate the wrenches on the left and right side, as shown.
- C. Press down the right side wrench and then lift it up on the left side.
- D. Handle with both hands and without letting go, pull the drawer out.
- E. Repeat for the second Drawer (B).

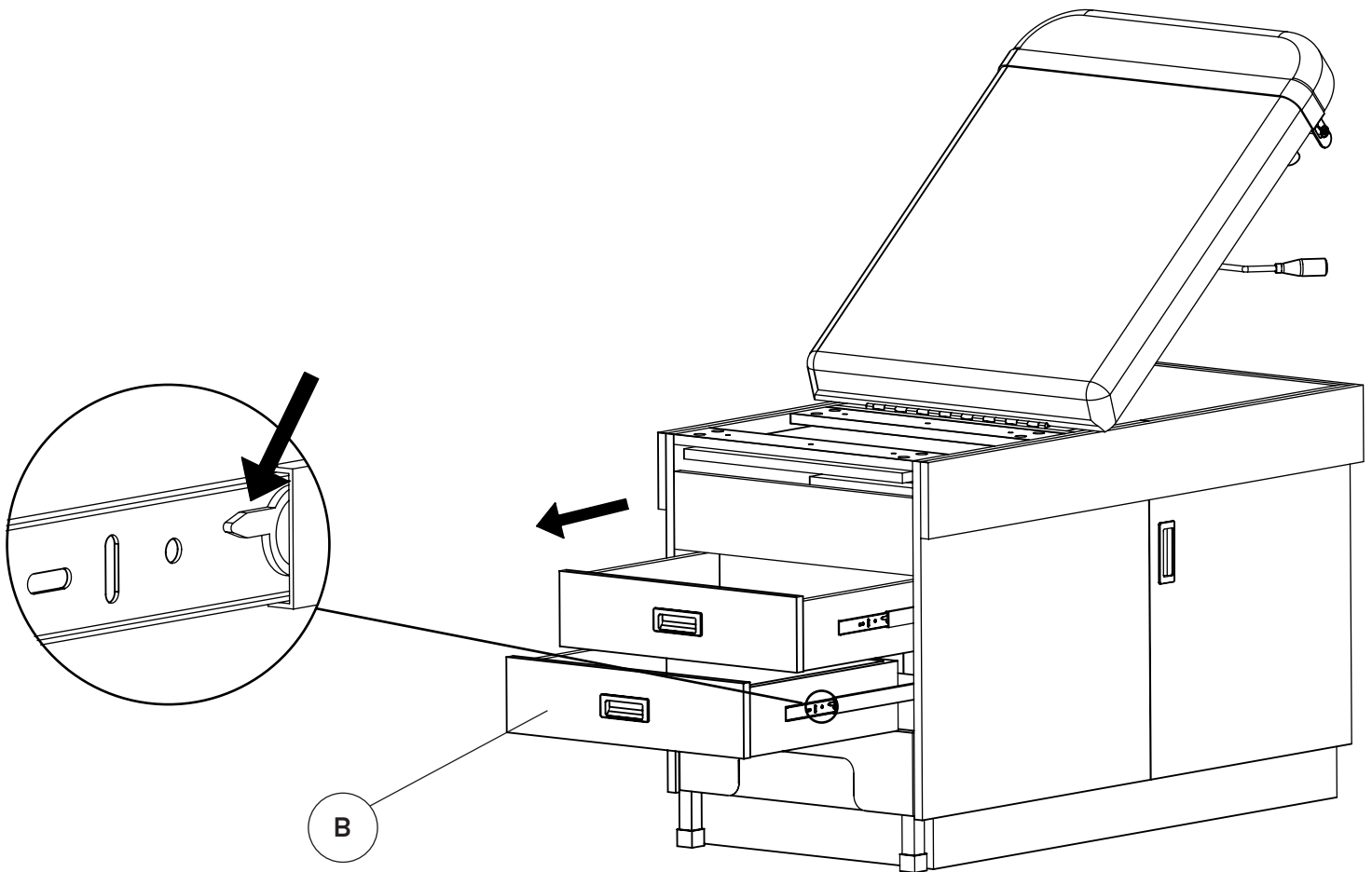

## Step 8:

- A. Fully extend the leg rest then place the Small Mattress (C) on top, as shown.
- B. Going from underneath where the Drawers (B) were, secure six M6\*30 Screw (PA4\*30) (2), three per board.
- C. Tighten them with a Philips head screwdriver.
- D. Push the leg rest back into the Table Base (F).

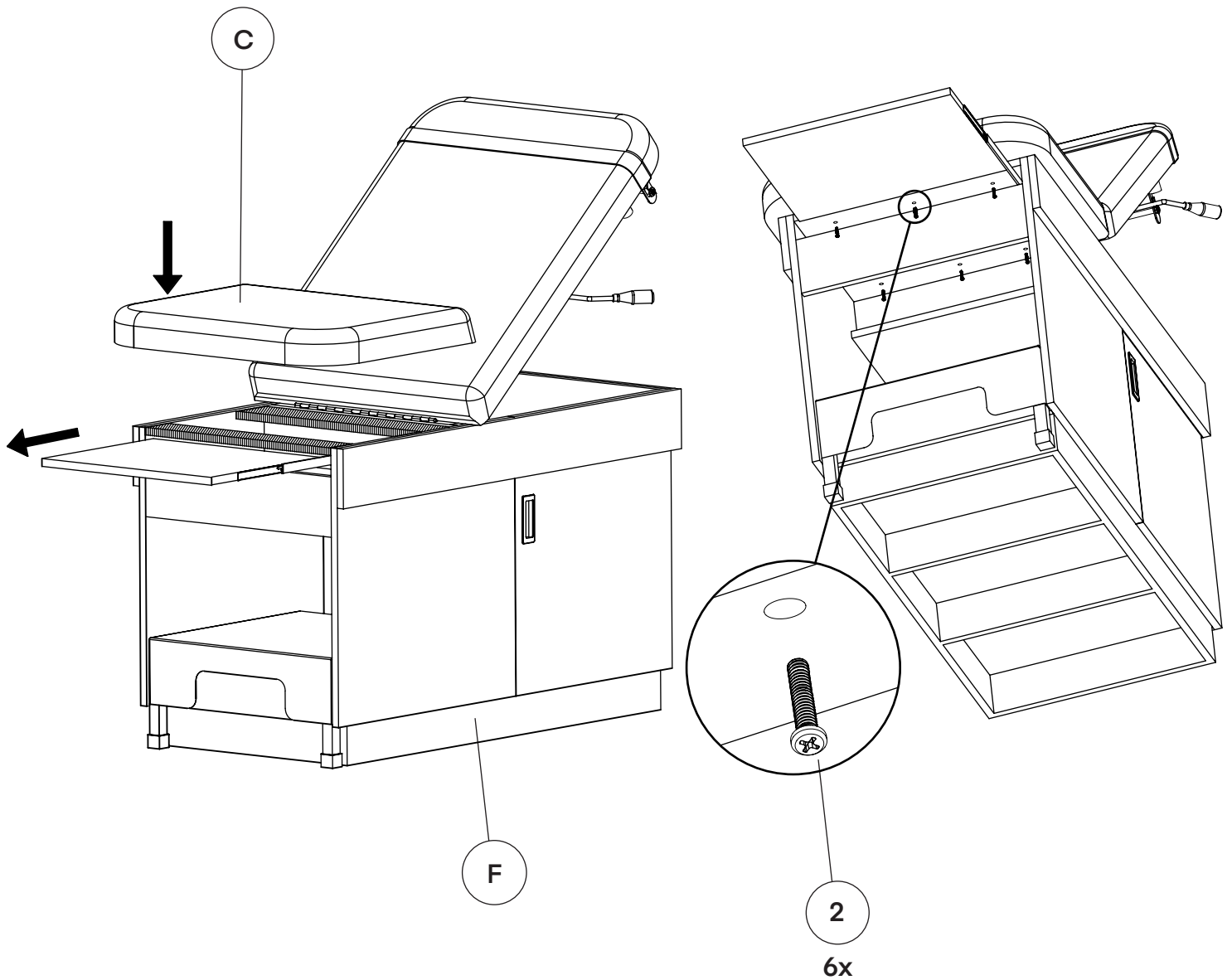


## Step 9:

**A.** Insert the Drawers (B) back into the Table Base (F).

**B.** Align the Drawers (B) with the guide rails on the base.

**a.** Do not operate the side wrenches.



**C.** Push both sides to the end of the guide rails and your exam table is now ready to be used.

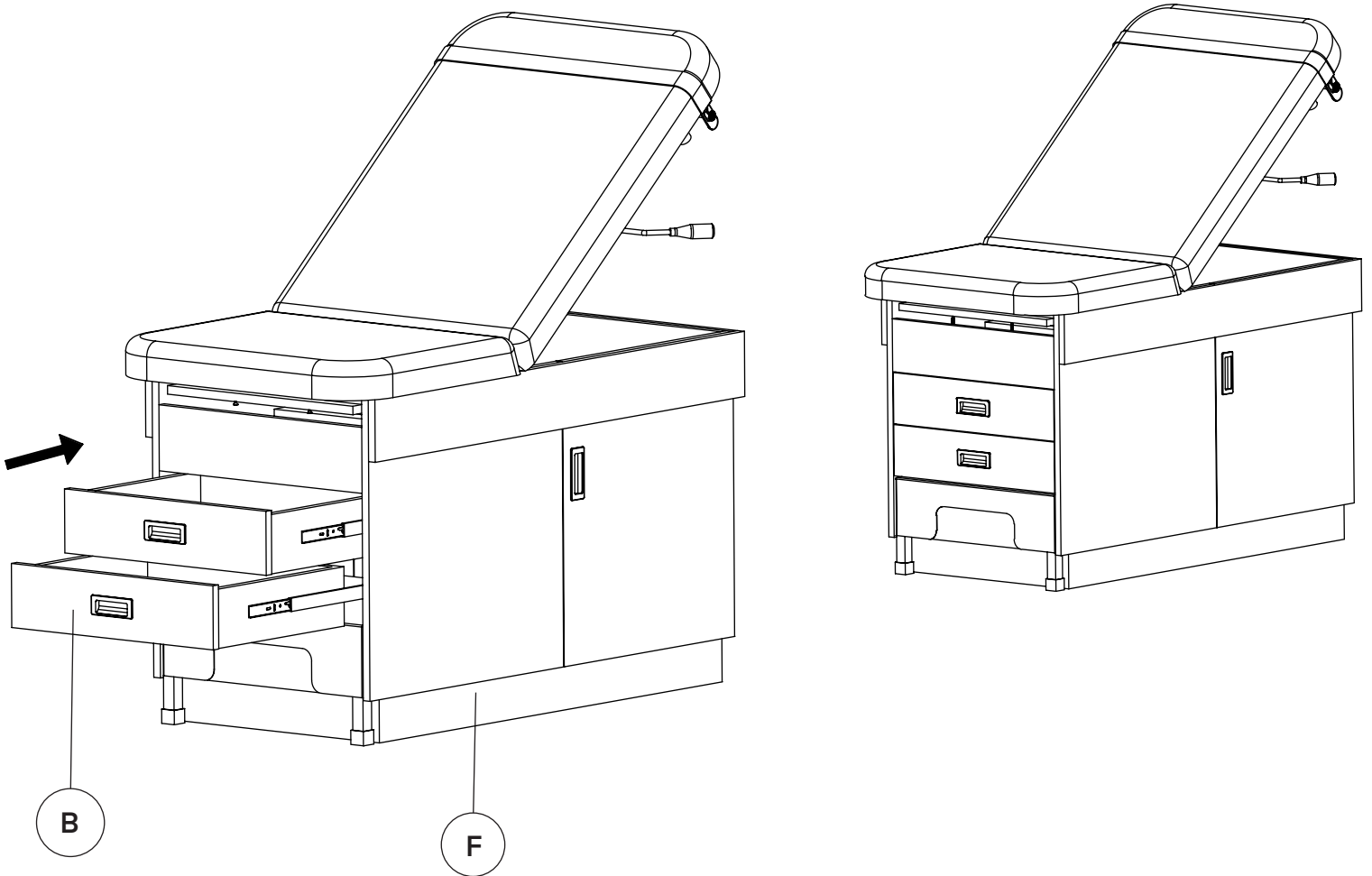

# Installing Paper Roll

## Step 1:

- A.** Remove the Paper Roll Holder (E) from the Left Support Element (13) and Right Support Element (14).
- B.** Slide your Paper Roll Holder (E) through your paper roll and then reinstall the Paper Roll Holder (E) onto the Left Support Element (13) and Right Support Element (14).
- C.** Unroll your paper, bring it over the top of the mattress and slide it underneath the Paper Cutter (A).

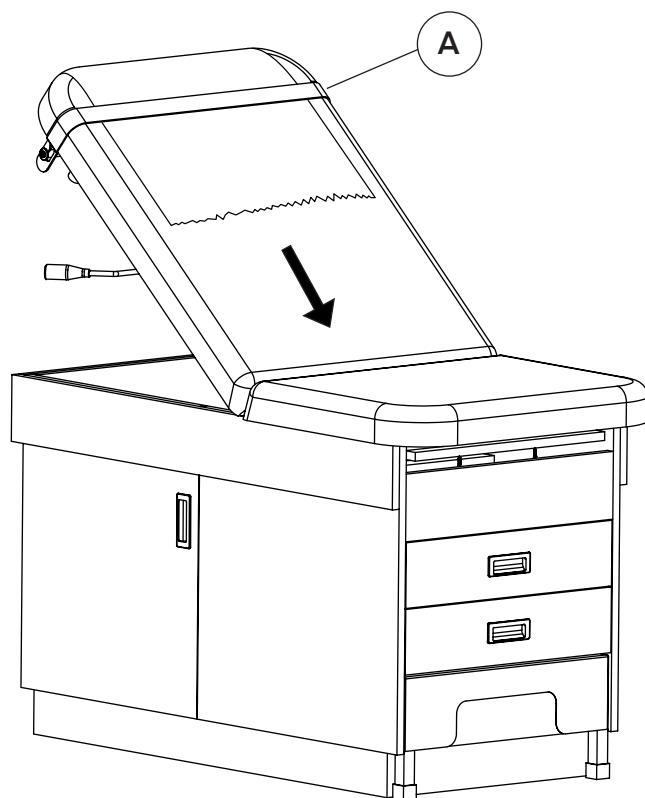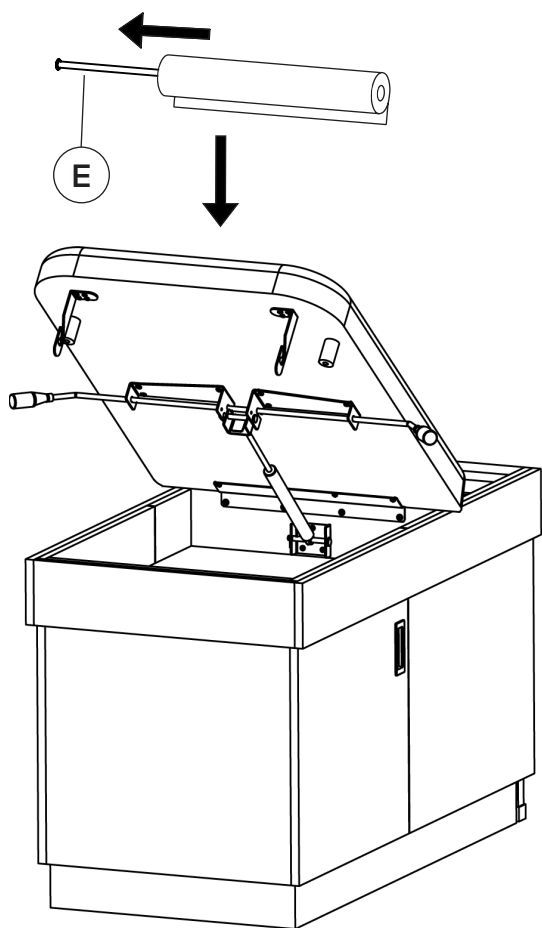

# Warnings

**Warning:** **DO NOT** lift the exam bed by the mattress. Lift it only by the side panels or the footrest rails.


**Warning:** To prevent injury, periodically check to see if the pull-out step stool's rubber feet are in good condition.

**Caution:** **DO NOT** sit on the pull-out leg rest or the head end of the table. They are not designed to support full body weight and excessive force. If you sit on the pull-out leg rest or the head end of the table, you could cause serious damage to the table or injure yourself.

**NOTE:** Our exam table is engineered for medical examinations use only. Anyone utilizing the table should be accompanied by a trained medical professional: never leave anyone on the table unattended. The weight and force exerted on the exam table should not exceed 500 lbs. Using the exam table for unintended purposes could result in damage to the table and/or serious injury to yourself and your patient.

# Warranty

1. Adir Corp (the "Company" or "Tiger Companies") warrants to the purchaser that the product will be free from defects in workmanship and materials for a period of 1 year from the date of purchase. The warranty period is not extended if we repair, replace, exchange, or provide a refund for the product (as determined in our sole discretion). We may change the availability of this limited warranty at our discretion, but any changes will not be retroactive.
2. These warranties are not assignable or transferable to any other person, including, without limitation, any subsequent owner or other transferee of the product.
3. This Warranty does not apply to: (a) damage caused by misuse, tampering, abuse, neglect, or accident; (b) damage caused by improper installation, modification, or service; (c) alteration of the serial number; or (d) use that violates the instructions furnished by the Company will void this warranty.
4. The sole responsibility of the Company shall be limited to (a) the repair or replacement (in its sole discretion) of any component of the product which fails to conform to this; (b) a refund the purchase price of such product (in its sole discretion), at no cost to the purchaser for the period of the warranty, or (c) an exchange for a similar product, comparable in function and price.
5. Contact the Company directly at 1-(800)-805-1790 to obtain service under this warranty. If it becomes applicable to send a defective product to the Company, a Return Authorization Number must first be obtained from the Company. In order to obtain service under this warranty, purchaser may be required to provide the Company with the following items (a) proof of purchase, (b) photographs and or videos (of the damage, and (c) a written testimonial describing the defect.
6. Products shipped without prior Return Authorization and Return Authorization Number may not be accepted, and the Company will not be responsible for their disposition and/or cost of return to the owner.
7. The Company will not assume any responsibility for any loss or damage incurred in shipping and or delivery.
8. The product(s) must be returned within 14 calendar days of receiving the return authorization from the Company and must include the original proof of purchase for the warranty to be honored.
9. Any implied warranties that the purchaser may have are limited to the duration of the warranties described above. There are no further warranties that extend or apply beyond the face hereof, and the Company expressly disclaims and excludes any and all warranties of merchantability or fitness for a particular purpose. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
10. THE REMEDIES DESCRIBED ABOVE ARE YOUR SOLE AND EXCLUSIVE REMEDIES AND THE COMPANY'S ENTIRE LIABILITY FOR ANY BREACH OF THIS LIMITED WARRANTY. OUR LIABILITY SHALL UNDER NO CIRCUMSTANCES EXCEED THE ACTUAL AMOUNT PAID BY YOU FOR THE DEFECTIVE PRODUCT, NOR SHALL WE UNDER ANY CIRCUMSTANCES BE LIABLE FOR ANY CONSEQUENTIAL, INCIDENTAL, SPECIAL OR PUNITIVE DAMAGES OR LOSSES, WHETHER DIRECT OR INDIRECT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATIONS OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE LIMITATION MAY NOT APPLY TO YOU.
11. This warranty gives you specific legal rights, and you may also have additional rights which vary from state to state.

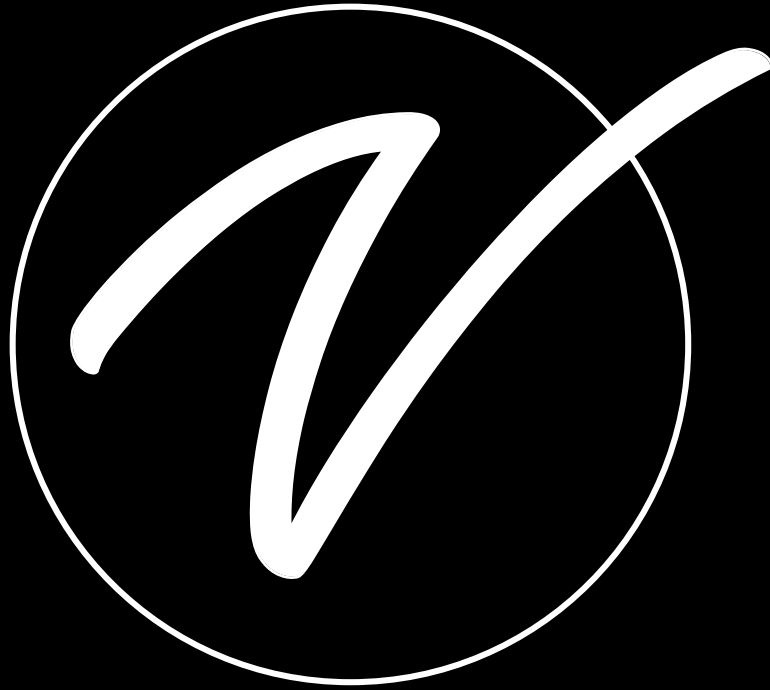

**ADI996-FBASE-SERIES**

© 2023 Adir Corp

**[adircorp.com](http://adircorp.com)**