

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

upholstered, wheeled
drop arm commode



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

commode assembly instructions and precautions



FIGURE 1

assembling commode

1. Before assembling seat, allow toilet seat to reach room temperature if exposed to cold. This will help prevent the seat clamps from cracking while being assembled onto commode back cross brace.
2. Insert Back support into frame.
3. To drop arm, depress release button on arm and gently allow to drop until reaching arm stop.
4. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the rail brackets). Push down on seat until clamps snap firmly into place.
5. When removing the seat grasp the back of the seat and lift.
6. Install commode bucket by placing in bucket frame. Reverse to remove.

additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Ensure ALL screws, nuts and/or bolts are tightened.
5. Users with limited physical strength should be supervised or assisted while using commode.