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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

bathtub safety rail

EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation instructions

1. Loosen adjustment bolts A and B until rail is wide enough to fit tub. Be sure to wash tub where rail is to be applied, and wipe dry.

All contact surfaces must be dry and free of soapy residue to assure secure attachment.

2. Place unit on tub with adjustment bolts outside tub and tighten with wrench.
3. Be sure to tighten both bolts A & B securely.

Safety Rail often requires retightening from time to time. Check periodically before use and retighten as needed.

CAUTION:

Please use extreme care when installing on fiberglass tubs. DO NOT OVERTIGHTEN as the clamp may crack the fiberglass.

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© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.