

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2005 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

bariatric free
standing trapeze



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

biatric free standing trapeze

assembly instructions

Assembly Instructions:

1. Remove contents from carton.
2. Attach slide plate on bottom of right and left base with bolts. (1)
3. Insert 4 base caps in front and rear of base. (2)
4. Attach wheels with axle bolts on left and right base. (3)
5. Slide mid-post on right and left base until lock pins appear through mid post. (4)
6. Slide boom (5) over mid post (4) while holding lock pin. (6) Once attached rotate boom until straight. (Lock pin until “clicks” into position).
7. While depressing lock pin on top of boom slide collar over boom. (7) (Be sure to slide collar past lock pin).
8. While holding top of chain/trapeze (8) inside collar, slide trapeze lock (9) through collar and “S” hook on chain. Turn trapeze lock until tight.
9. Insert boom cap onto boom.

