

2 year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for 2 years from the time of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

dial-a-speed tabs

CE 1400

variable resistance tabs for use with all anterior and posterior safety rollers and gait trainers



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel *Re/hab*

www.drivemedical.com



drive
WWW.DRIVEMEDICAL.COM

installation

1. Remove nut.
2. Insert bolt into Component A & B.
3. Insert bolt and Component A & B into the fork assembly.

NOTE: Screw head should face the wheel.

4. Mount nut on the outside of fork assembly.
5. Tighten nut and bolt.

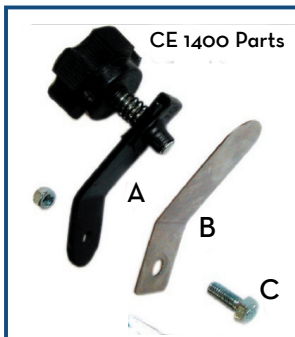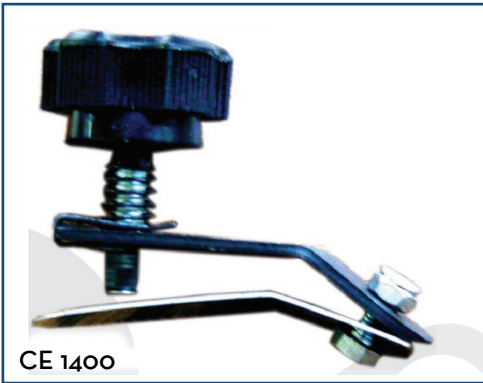


adjustment

Turn knob clockwise to increase tension on wheel and counter-clockwise to decrease tension on wheel.



mounted on posterior safety roller

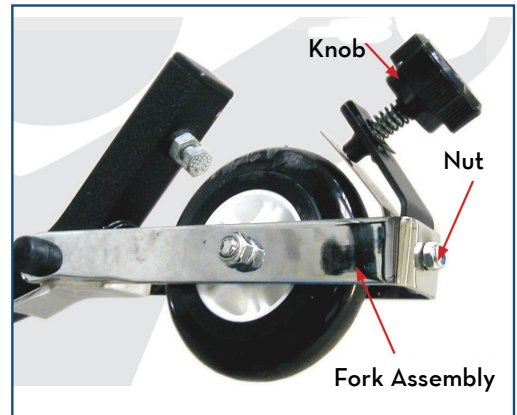

mounted on anterior safety roller