WelchAllyn®

Ophthalmoscopes, Otopscopes and No. 20000 Ootoscope/Throat Illuminator
Part No. 705710 Rev. D

For guaranteed performance, use only genuine Welch Allyn lamps.

Operating Instructions for No. 20000 Ootoscope/Throat Illuminator

To attach Ootoscope/Throat Illuminator to power source (Figure 1):

Align cutouts in the Ootoscope/Throat Illuminator base with lugs on power source. Push in slightly and turn 90°. The Ootoscope/Throat Illuminator is now ready to use. To remove, reverse this procedure. The Ootoscope/Throat Illuminator fits all 3.5 v. Welch Allyn power sources.

To use Throat Illuminator (Figure 2):


- Grasp the ootoscope section with the left hand.
- With the right hand, grasp the throat illuminator's collar at A (under the silver ring).
- While holding the throat illuminator collar stationary, twist the ootoscope section counterclockwise at A until the ootoscope section becomes loose.
- Slide the right hand down to the power handle, grasp it and turn it clockwise while keeping the ootoscope section stationary, until ootoscope section is removed. The throat illuminator is now ready for use.

To replace ootoscope section, reverse the steps.


Note: It is important to note that while the throat illuminator section provides an excellent source of general illumination, more than momentary direct skin contact should be avoided due to possible heat build up.

Lamp Replacement Instructions for Welch Allyn Otopscopes

Fiber Optic Otopscopes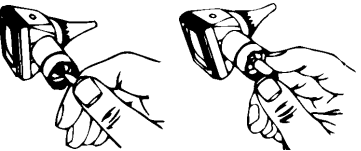

Conventional Otopscopes

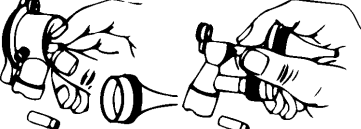Lamp Replacement Instructions for Welch Allyn Ophthalmoscopes

Lamp Replacement (Figure 3):

Caution: Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eyeglasses when handling. Be sure power is off when replacing lamp. Dispose of lamp with care.

CAUTION: Lamps may be hot. Lamp should be allowed to cool before removal.

Cleaning Recommendation

Repairs and cleaning (with the exception of lamp replacement) beyond what is specified in the general clean instructions section should be performed at the factory. Do not apply alcohol, chemicals, or water to the mirror or lenses.

Note: Excess solution entering the optical assembly could damage internal components. Use caution to ensure cloth is not saturated with solution.

The ootoscope and ophthalmoscope instrument heads may be cleaned with a dry cloth. Windows may be cleaned with a cotton swab dipped in alcohol or lens paper.

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